

LOTUS TRANSPOTER
HCM9062F
Aluminum Transport Wheelchair



1. Introduction

Powder coating aluminum frame
Single cross braces
Flip back desk armrest
Swing in and out footrest
8"x2" PU front castor
12" rear wheel with PU solid tire
Nylon with sponge
Backrest half folding
Attendant brake
With small wheels

Specification

N.W./ G.W.: 10/ 12 Kg

Weight capacity: 120 Kg

Overall Dimensions: 97 (L)* 63.5 (W)* 92 (H) cm

Seat Width: 40 cm, 45cm, 51cm

Seat Depth: 41 cm

2. Mounting



3. Use

3.1 Unfolding the Wheelchair



Figure 1

Hold the handles with both hands to separate the frame slightly. Push down both sides of the seat upholstery to flatness lightly by hand, so that the wheelchair has been unfolded. (Figure 1)

3.2 Folding



Figure 2

First turn over both the footplates, lift up both sides of the seat upholstery lightly with hands, and then it can be folded. (Figure 2)

3.3 Use of brake



Figure 3

The user holds the brake handles with both hands when the wheelchair stops, and then moves the brake handles forward to self-locking condition, to make the rear wheels stop completely to keep still. The process for braking is completed. (Figure 3)

3.4 Use of armrest



Figure 4

When detaching the armrest, first press the armrest fastener with one hand to separate it from the front bracket of seat tube, then you can flip it back. When assembling the armrest, make the armrest fastener aim at the position of the front bracket of the seat tube and press lightly to self-locking fastener. (Figure 4)

3.5 Use of legrest



Figure 5

To mount the footrest, positioning holes to fit with the sides of the footplate. Fit the footrests and rotated inwards until the closing perfectly. To remove the footrests, squeeze the handle rotate footrest outward and lift them up. (Figure 5)

4. Caution

4.1 General precautions

- a. This wheelchair has been designed for Indoor use of vehicles of any kind. Always transfer to the user of the Chair in an approved seat by the vehicle manufacturer and use all available Safety restraints.
- b. Do not use this wheelchair in Roads, streets or any Surface traffic.
- c. Don't try to operate in inclined planes of 10% or more, without adequate support.
- d. Make sure to fully Activate the Wheel Brakes when getting on or off the Chair, or when you're in an elevator or Lift for wheelchair. The Wheelchair must be on a Flat Surface, stable and level.
- e. When you get on or off the wheelchair, do not climb or put Weight on the foot. When trying to extend the arms on the back, do so

only until his arm can be extended without changing your sitting position.

f. Do not stand in the wheelchair.

4.2 Stability

Wheelchairs are designed to be safe and stable while the center of gravity is maintained during normal use. You must take precautions if you want to perform activities that require a change of the center of gravity. Should never try to achieve something that is not available to any way, bowing out of the Chair. If you need to lean or Lean on the Chair, take care that the front Wheels pointing Forward, so as to extend the base surface of the Wheel, to stabilize the Chair.

4.3 Ramps

Make sure your own capacity constraints and force before attempting to upload a Ramp. You can ask a professional qualified the right skills to go up or down a Ramp. It is advised to help when going up or down slopes over 10. If you must stop on a Slope, avoid Sudden movements and Quick when you move. A Move can cause instability due to the angle of the Chair. The use of The Brakes could result in accidental Brake Brakes, an uncontrolled movement or a sudden blockage. Try not to change direction down a Ramp, could cause instability and lead to the overturning of the Chair.

4.4 Maintenance of the Chair

Always make sure that the chair is dried after use to prevent corrosion. The chair should be cleaned frequently, using warm water and a mild detergent. Dry after cleaning. All the metal parts can be cleaned with Wax for cars or similar products. Do not use Abrasive Cleanser that can scratch the finish.

Among the uses the wheelchair must be folded for Storage. The Chair can be folded to approximately half its width.

4.5 Security

Manual wheelchair has a legal guarantee of two years from the invoice against manufacturer defect. The warranty does not cover the wear of wheels, seats and backrests, Brakes, so as problems caused by improper use or maintenance had not been indicated. In this case the transport costs for compensation shall be borne by the customer.

The Wheelchairs has been tested and certified in accordance with CE

requirements and BSEN 12183.

